

Erase blemish!



Laser-Free Treatment system  
**BEAUTY CLEANSER**

Eraseblemish!



## INDEX

1. Components
2. Beauty Cleanser Indication
3. What is beauty cleanser?
4. Beauty cleanser principle
5. Preparation guide
6. Treatment
7. How to use
8. Clinical trial pictures
9. Benefit of Beauty Cleanser

# 1. Components

Package box



Cover

Inbox

Main Device



Adaptor

Special Tip  
2EA



## 2. Indication



- ✓ Only at one treatment, you can find immediate and permanent effect on your skin.

### 3. What is Beauty Cleanser?


Beauty Cleanser is for taking out of milium, skin tag, sun spot, syringoma and verruga from the skin with simple way by Beauty Cleanser.

- ✓ Using RF, Beauty Cleanser works precisely to small part of your skin
- ✓ Minimally invasive, painless, no downtime
- ✓ Minimized the damage to skin texture
- ✓ Without high-priced laser equipment, save the skin defect at a reasonable cost

## 4. Beauty Cleanser Principle

- ✓ Beauty cleanser used in this method to treatment emits ultra high frequency radio waves which exactly vaporise the cells of the mole but leaves the healthy skin cells below with little or no tissue damage to allow very quick healing process and minimal risk of scarring.
- ✓ RF spot removal aesthetic device promote epidermal cell necrosis through the Beauty Cleanser. It make coagulation, can remove the blemish on the skin surface in an instant, such a wart, freckles, granulation, moles and so on, without Bleeding, no scarring, and the effect is obvious.


## 5. Preparation Guide

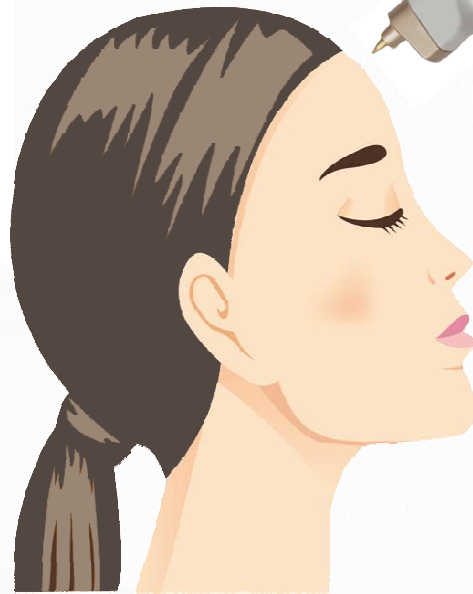

- 01** Clean the treat region before procedure.
- 02** Apply alcohol after cleansing.
- 03** Apply local anesthesia.(**Nexcaine 9.6% cream\***)  
Do not exceed 20min.

## 6. Treatment

	Mode	Description
LOW	1	• Wrinkle
	2	• Acne
	3	• Cherry Hemangiomas • Sun spot
	4	• Age spot
HIGH	5	
	6	
	7	
	8	• Skin Tag • Wart • Spot (Deep)

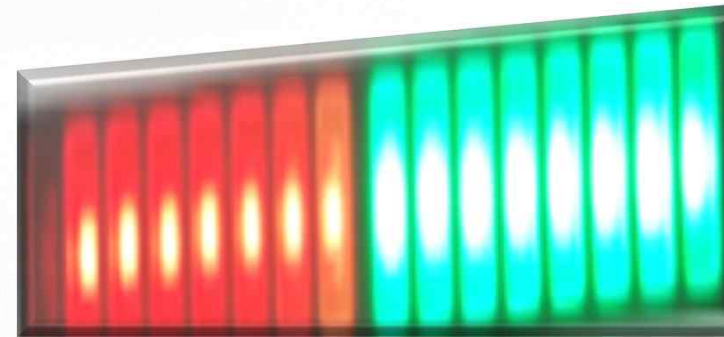


**Power**      **FreQ**

- ✓ Two control system : Power & Frequency
- ✓ High Frequency : reduce pain & scar
- ✓ POWER : Stable Energy and simple using

Beauty Cleanser has Mode 1 to 8. Intensity of the RF increase as the number increase. You can choose the proper Mode for each Indications. Refer to following chart.



## 7. How to use?

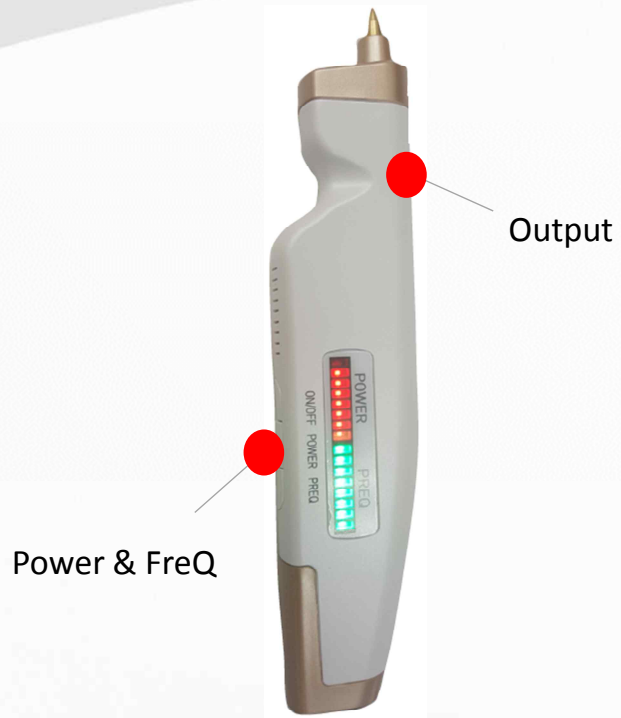

- ✓ Remove the paper wrapping on a needle and attach the needle to a main device.
- ✓ Connect an adapter to the main device.
- ✓ Press Power button(On/Off) to turn on for 2-3 seconds.
- ✓ Set Power and FreQ.
- ✓ After setting power and Freq just press button of “Output” to ready .
- ✓ When you press the output button energy is coming out.
- ✓ After finishing Press Power button(On/Off) a few second to turn off.

## 8. Clinical Trial Pictures Before & After

## Clinical Trial Picture Before & After



## Clinical Trial Picture Before & After



## Clinical Trial Picture Before & After



## Clinical Trial Picture Before & After



## 9. Benefit of Beauty Cleanser

- 1. Easy to use**
- 2. Safe and Effective**
- 3. Less pain**
- 4. Precise treatment**
- 5. No down time**
- 6. Reasonable price**

# END

[www.pronexx.com](http://www.pronexx.com)

Sales contact email: (Kathryn Kang) [kathryn@pronexx.com](mailto:kathryn@pronexx.com)

Head office telephone : +82 (0)2 572 1331